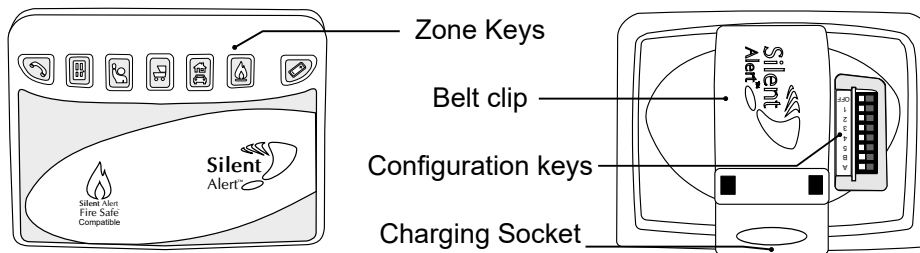


Pager recovery guide

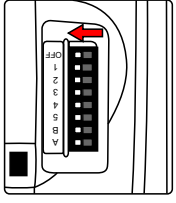
Silent Alert

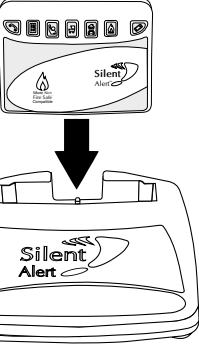


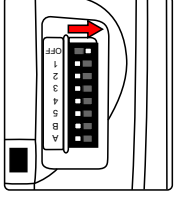
Any questions? Call us on free phone 0800 387 397

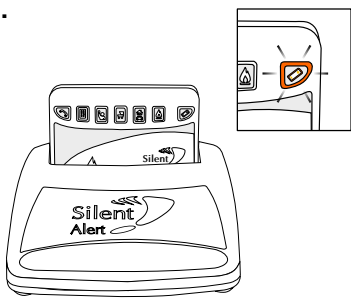


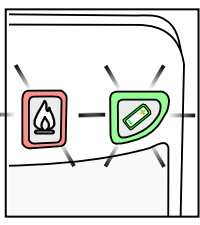
If the Pager is not recharged for an extended period of time the charge in the internal battery may become so low it may struggle to switch on. The following procedure should recover the battery.

- 

1. Move the ON/OFF switch to the OFF position (to the left towards off).
- 

2. Insert the Pager into the charging slot and charge for 1 hour. Do not switch the Pager on until this time has elapsed.
- 

3. Move the ON/OFF switch to the ON position (to the right away from off).
- 

4. The Battery Key should light green. AS SOON AS IT DOES press the battery key to cancel the start up routine.
- 

5. Do not let the start up light routine run as this will further drain the battery.
6. Leave the Pager on charge in this state for at least 24 hours.

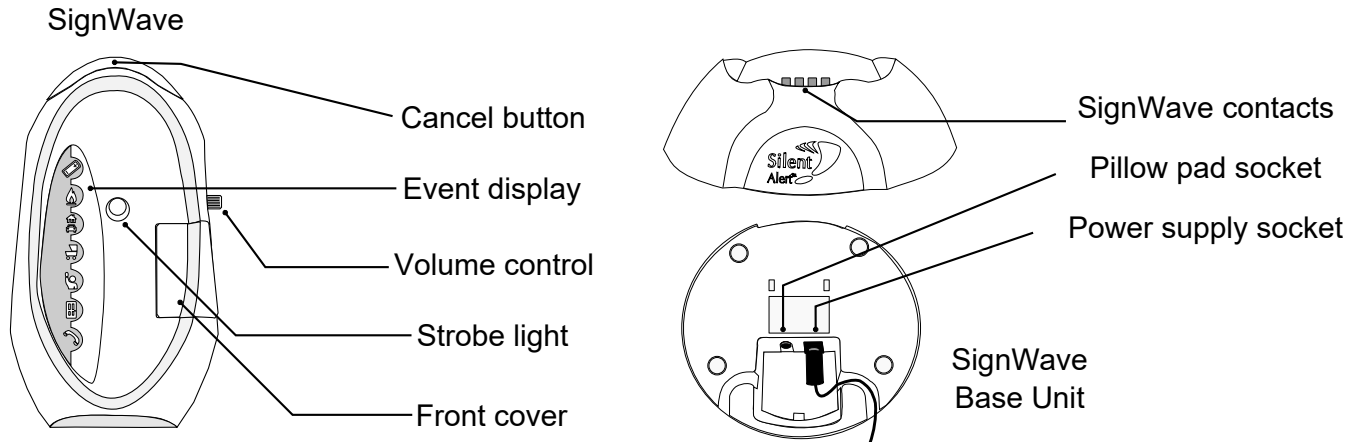
The Pager should now be fully charged and back to normal.

SignWave recovery guide

Silent Alert



Any questions? Call us on free phone 0800 387 397



If the SignWave is not recharged for an extended period of time the charge in the internal battery may become so low it may struggle to switch on. The following procedure should recover the battery.

- 1.**

Insert a small flat blade screwdriver into the slot on the front cover.

Push gently and the cover will pivot outwards
- 2.** Move the On /Off key to the RIGHT using the edge of the front cover.
- 3.**

Dock the SignWave onto the base. charge for 1 hour.

Do not switch the SignWave on until this time has elapsed.
- 4.**

Move the On /Off key to the LEFT using the edge of the front cover.

The SignWave will switch on and will indicate its battery state via its battery light.
- 5.**

If the battery light shines green the unit has recovered.

If the battery light flashes orange every 40 seconds the unit has recovered but has a low battery. The SignWave may alarm in this state, this is normal.
- 6.**

If the battery light flashes orange 10 times it is still very low and the SignWave is shutting down. Try repeating steps 1 - 4 again.

The SignWave should be left for 24 hours to fully charge. The unit should be placed on the base each night to recharge & can be left on the base indefinitely if required. It can then be taken off the base and used as required.