

Alarm Clock Charger

CH3A-2205-IN

Silent Alert Alarm Clock Charger Explained


Published January, 2020



Alarm Clock Charger



Basic Bedside Charger

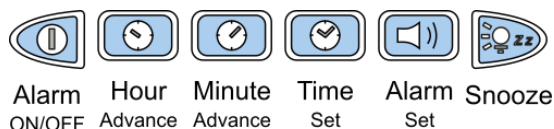

Fully Functional Alarm Clock

There are two pager charger options, the basic bedside charger and the alarm clock charger that features a fully functional digital alarm clock. If a signal is received when the pager is docked a pillow pad connected to the charger will vibrate to wake the user. The vibration pattern is different for each event and matches those of the pager. The same pillow pad vibrates when the clock alarm activates.

Handy Tip – Fault condition (Alarm Clock Charger only)

An orange light in the "Snooze" key and the clock display will flash alternately if the mains is switched off or there is a fault with the pillow pad. To clear the fault ensure the mains supply is on and the pillow pad is plugged in correctly. If the fault light continues to flash remove the pillow pad, if the flashing stops then the pillow pad is faulty and will need to be replaced.

Handy Tip – Alarm Clock Charger Shutdown

If the alarm clock charger is to be stored or is being transported, it is possible to shut the unit down, deactivating the fault warning lights and to conserve the backup battery. After the power has been switched off, simply press and hold the "Snooze" and "Alarm" keys together (last two keys on the right). You may have to repeat this a couple of times until the keys light in sequence from left to right, followed by all the keys lighting together briefly. This will put the charger into hibernation. To take the unit out of hibernation simply switch the power back on again.